SUNALE

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Support device for washing and showering. It is a technical aid that allows people with reduced mobility or disability, who cannot make conventional use of the shower, to remain seated during the bath. It is also an option to improve the quality of life of people who suffer from urinary incontinence.

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This product is a Class I medical device and complies with Regulation (EU) 2017/745 of the European Parliament and of the Council of April 5, 2017 on Medical Devices. For the disposal of the container and the product, strictly comply with the legal regulations of your community.

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Our products are guaranteed against any manufacturing defects for two years from the date of the invoice. For more information please refer to our general sale conditions.

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Any serious incident related to the product must be reported to the competent authority of the Member State in which the user and / or patient are established.

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SELF PROPELLEING AND TRANSPORT SHOWER WHEELCHAIR

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The rear wheels can be fully locked using the foot brakes or hand brakes depending on which rear wheels are installed.

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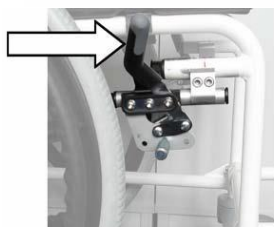
Do not attempt to reach any objects in any of the following cases:

1. If it implies that the user has to advance forward in the seat
 2. If the user has to pick them up from the ground
- Una vez ajustado el reposapiés, es importante comprobar que la altura e inclinación fijada no cause fricción entre la parte trasera de las piernas y la tapicería de la silla. After adjusting the footrest, it is important to check that the height and inclination set does not cause friction between the back of the legs and the upholstery of the chair.
 - Los reposapiés deben quedar separados del suelo por lo menos 65 mm para evitar posibles caídas. The footrests must be, at least, 65 mm above the floor to avoid possible incidents.
 - Se debe bloquear los frenos cuando el usuario se sienta o levante de la silla. Lock the brakes when the user sits down or stands up.

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The use of the chair is not suitable for people with strong perception disorders, balance disorders or an inability to sit.

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The life span of this item is influenced by their conditions of use and storage and by the regularity of their maintenance. By imperatively respecting the data provided in this instruction manual, it is possible to keep the item in perfect condition for 24 months.

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REF	Referencia del producto / Product reference	Fecha fabricación / Manufacturing date	
MD	Producto Sanitario / Medical Device	LOT	Número de Lote / Lot Number
CE	Certificado CE / EC certificate	Manual de Instrucciones / Use for instructions	
Fabricante / Manufacturer		Precaución / Precaution	
Importador / Importer		EC REP	Representante Europeo Autorizado / European Authorised Representative